

Igloo[®] Wound Irrigation Shield

INTENDED USE

The Igloo Wound Irrigation Shield is intended to be used during wound irrigation to protect the user from backslash as well as controlling the effluent runoff.

INDICATIONS

The Igloo Wound Irrigation Shield is indicated for use during the general cleansing of wound sites. The Igloo Wound Irrigation Shield may be used on all ages of patients.

CONTRAINDICATIONS

- The Igloo Wound Irrigation Shield is not indicated for the cleansing of deep or puncture wounds.

⚠ WARNINGS

- Do not use if the package is open or damaged.
- For deep or puncture wounds, animal bites or serious burns, consult a healthcare professional.
- Single-use only.
- To reduce aerosolization, keep the Igloo Wound Irrigation Shield in contact with the skin during the irrigation procedure.

USER INSTRUCTIONS

1. Prep and drape the wound in a sterile fashion. Prepare a basin with sterile saline or other desired irrigant solution.
2. Fill a standard luer lock syringe or other delivery device with the irrigation solution.
3. Attach the Igloo Wound Irrigator to the tip of the syringe or other delivery device.
4. Place the Igloo Wound Irrigator over the wound, lightly touching the skin of the patient. For large, spacious (or chronic) wounds, place Igloo just above the wound surface.
5. Irrigate the wound in the usual fashion; syringe may need to be refilled one or more times as needed.
6. After use, discard the Igloo Wound Irrigator appropriately.



**SINGLE
USE ONLY**

R_x ONLY

**MADE IN
THE USA**



STERILE R

