Lighted Articulating Ear Curette™

INSTRUCTIONS FOR USE

Obelis s.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03
E-Mail: mail@obelis.net

R_x ONLY

Bionix LLC
1670 Indian Wood Circle
Maumee, Ohio 43537, United States
Phone: +1 419.727.8421
Fax: +1 419.727.8426
www.Bionix.com

ASSEMBLED IN THE USA

**Importer, Importeur, Importör,
Importador, Importateur,
Importeur, Importør, Dovoze,
Maahantuoja, Importatore,
Εισαγωγέας, Importuotojas:**

MedEnvoy
Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague
The Netherlands

Lighted Articulating Ear Curette US Patent #: 8,728,107

©2025 Bionix LLC | 1670 Indian Wood Circle | Maumee, Ohio 43537 | United States
Phone: 800.551.7096 (+1 419.727.8421) | Fax: 800.455.5678 (+1 419.727.8426)
Email: bionix@bionix.com (international@bionix.com) | www.bionix.com
RM95-0011 | 2025/11/23 | Rev. 02

Lighted Articulating Ear Curette™ IFU

English	3
Deutsch (German)	4
Español (Spanish)	5
Français (French)	6
Svenska (Swedish)	7
Polski (Polish)	8
Česky (Czech)	9
Português (Portuguese)	10
ελληνικά (Greek)	11
Nederlands (Dutch)	12
Suomi (Finnish)	13
Italiano (Italian)	14
Lietuvių k. (Lithuanian)	15
Dansk (Danish)	16
Bokmål (Norwegian)	17
български (Bulgarian)	18
Română (Romanian)	19

ENGLISH

Intended use: The intended use of the Lighted Articulating Ear Curette is to remove cerumen and/or foreign bodies from and illuminate a patient's ear canal without causing injury.

Indications for use: The Lighted Articulating Ear Curette is indicated for use in cases of impacted cerumen and/or foreign bodies occluding the ear canal.

Assembly:

- To assemble using the Original Light Source:** Insert the end of the tool into the Light Source by aligning the pegs on the end of the tool with the grooves inside the Light Source. Push down and **FIRMLY** twist clockwise until rotation stops. This will lock the tool into place and activate the light.
- To assemble using the Rechargeable Light Source:** Align the end of the tool with the threaded opening of the Rechargeable Light Source. Turn the Light Source clockwise until secure. Do not overtighten.
- To attach the Magnification Lens:** Snap the lens onto the neck of the Light Source so that the top of the lens angles toward the tip of the tool.

Disassembly:

- To disassemble from the Original Light Source:** Push the tool into the Light Source and rotate counterclockwise. The light will go off when the tip is removed.
- To disassemble from the Rechargeable Light Source:** Simply twist the Light Source in a counterclockwise direction until the Light Source has loosened.

Instructions for Cerumen Removal - Using the Lighted Articulating Ear Curette™ with Magnification

The Lighted Articulating Ear Curette offers an effective approach to cerumen and foreign body removal, with an articulating, lighted tip that can be made to flex and firmly, yet gently, remove obstructing wax or objects.

To use the Lighted Articulating Ear Curette:

- Position the patient comfortably on the examination table. Smaller children may be restrained on their parent's lap or shoulder during the procedure.
- Using an otoscope, examine the ear canal noting the presence and location of any obstructing cerumen or foreign body.
- Straighten the ear canal by gently pulling the ear up and away from the patient. Position the tip of the Lighted Articulating Ear Curette adjacent to the obstruction or behind it if there is room between the obstruction and the ear canal wall.
- Grasp the handle of the Lighted Articulating Curette in your palm placing your index finger in the trigger and gently pull back. This will cause the tip of the curette to bend.
- Hold the tip in the flexed position and gently roll the obstruction from the ear canal.
- Use an otoscope to re-examine the ear canal, noting the

presence and location of any remaining obstruction.

- Repeat the above procedure as needed to until the tympanic membrane can be adequately visualized, or the foreign body has been removed.
- Remove the Light Source and discard the curette tip.

⚠ CAUTION: Discontinue curettage immediately if bleeding, irritation, or other trauma to the ear canal or tympanic membrane occurs.

⚠ CAUTION: This product is a spring-loaded device; improper assembly may cause ejection of the ear curette.

⚠ CAUTION: DO NOT hold device by Light Source as accidental ejection of curette may occur if not assembled properly. Special Note: The Lighted Articulating Ear Curette is intended for single-patient use. Discontinue use of the curette if the hinge no longer functions. Re-use of the curette can cause the hinge to stop functioning. Use of alcohol wipes or cold sterilization can also negatively affect the functioning of the curette. The Light Source and Magnification Lens are designed for multi-procedure use.

⚠ WARNING: Cross-contamination risk. Do not reuse disposable curette tips as this may spread contamination from one patient to another patient.

Medical Device Reporting: Notice to Users and/or Patients in EU: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Clinical Benefits: Indicated in Intended Use.

Contraindications: There are no absolute contraindications for cerumen removal.

Residual Risk: Risk associated with the use of this product has been reduced as far as possible, but the product cannot completely eliminate potential patient or user harm arising from the following:

- Harm from mechanical hazards
- Harm from misuse, or use error
- Harm from unanticipated origins

	Medical Device		Do Not Use if Package is Damaged
	Manufacturer		Do Not Reuse Single-Use Device
	Date of manufacture		Prescription Only or "For Use by or on the order of a licensed medical professional"
	Lot Code		Consult Instructions For Use (IFU)
	Reorder Number		Caution
	European Conformity		Warning
	Authorized representative in the European Community		

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