Lighted Forceps for Foreign Body Removal

INSTRUCTIONS FOR USE

Obelis s.a
Boulevard Général Wahis 53
1030 Brussels, BELGIUM
Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03
E-Mail: mail@obelis.net

R_x ONLY

Bionix LLC
1670 Indian Wood Circle
Maumee, Ohio 43537, United States
Phone: +1 419.727.8421
Fax: +1 419.727.8426
www.Bionix.com

ASSEMBLED IN THE USA

**Importer, Importeur, Importör,
Importador, Importateur,
Importeur, Importør, Dovoze,
Maahantuoja, Importatore,
Εισαγωγέας, Importuotojas:**

MedEnvoy
Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague
The Netherlands

Lighted Forceps for Foreign Body Removal US Patent #: 10092168

©2025 Bionix LLC | 1670 Indian Wood Circle | Maumee, Ohio 43537 | United States
Phone: 800.551.7096 (+1 419.727.8421) | Fax: 800.455.5678 (+1 419.727.8426)
Email: bionix@bionix.com (international@bionix.com) | www.bionix.com
RM95-0017 | 2025/11/23 | Rev. 03

Lighted Forceps for Foreign Body Removal IFU

CONTENTS

English	3
Deutsch (German)	4
Español (Spanish)	5
Français (French)	6
Svenska (Swedish)	7
Polski (Polish)	8
Česky (Czech)	9
Português (Portuguese)	10
ελληνικά (Greek)	11
Nederlands (Dutch)	12
Suomi (Finnish)	13
Italiano (Italian)	14
Lietuvių k. (Lithuanian)	15
Dansk (Danish)	16
Bokmål (Norwegian)	17
Română (Romanian)	18

ENGLISH

Intended use: The intended use of the Lighted Forceps is to remove foreign bodies from the ear canal and/ or the nasal passage.

Indications for use: The Lighted Forceps is indicated when there is a need for removal of leafy or soft foreign bodies in the external ear canal or nares.

Assembly:

- To assemble using the Original Light Source:** Insert the end of the tool into the Light Source by aligning the pegs on the end of the tool with the grooves inside the Light Source. Push down and FIRMLY twist clockwise until rotation stops. This will lock the tool into place and activate the light.
- To assemble using the Rechargeable Light Source:** Align the end of the tool with the threaded opening of the Rechargeable Light Source. Turn the Light Source clockwise until secure. Do not overtighten.
- To attach the Magnification Lens:** Snap the lens onto the neck of the Light Source so that the top of the lens angles toward the tip of the tool.

Disassembly:

- To disassemble from the Original Light Source:** Push the tool into the Light Source and rotate counterclockwise. The light will go off when the tip is removed.
- To disassemble from the Rechargeable Light Source:** Simply twist the Light Source in a counterclockwise direction until the Light Source has loosened.

Instructions For Foreign Body Removal

- Examine the ear or nose to determine the location of the foreign body.
- Assemble the Lighted Forceps for Foreign Body Removal according to instructions.
- Insert the Lighted Forceps tip until it reaches the foreign body. Once there, squeeze the spring grip to grasp the object and gently pull from the orifice.
- Repeat the exam and determine location of any remaining part of the foreign body.
- Repeat the procedure until foreign body is removed.
- Remove and dispose of the single-use forceps.

Disposal

Lighted Forceps should be recycled or disposed according to current local, regional, state, and/or federal laws and regulations pertaining to the safe disposal of medical devices and accessories. If the user is uncertain how to proceed, they should contact Bionix, LLC for guidance on safe disposal protocols.

⚠ CAUTION: Discontinue use immediately if bleeding, irritation, or other trauma to the ear canal or tympanic membrane occurs.

⚠ CAUTION: This product is a spring-loaded device; improper assembly may cause ejection of the forceps.

⚠ CAUTION: DO NOT hold device by Light Source as accidental ejection of forceps may occur if not assembled properly.

⚠ WARNING: Cross-contamination risk. Do not reuse disposable Lighted Foreign Body Removal Tool as this may spread contamination from one patient to another patient.

Medical Device Reporting: Notice to Users and/or Patients in EU: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Clinical Benefits: Indicated in Intended Use.

Contraindications: The following are absolute contraindications for use of the Lighted Foreign Body Removal Tool.

- Button or hearing aid batteries are indications for emergent ENT consultation
- Trauma to the ear canal or tympanic membrane

The following diagnosis/conditions are relative contraindications to the use of the Lighted Foreign Body Removal Tool:

- Tympanic membrane perforation
- Foreign body larger than the width of the Foreign Body Removal Tool jaw opening
- Foreign Body in contact with the tympanic membrane
- Pus or exudate in the ear canal obscuring view of foreign body

Residual Risk: Risk associated with the use of this product has been reduced as far as possible, but the product cannot completely eliminate potential patient or user harm arising from the following:

- Harm from mechanical hazards
- Harm from misuse, or use error
- Harm from unanticipated origins

	Medical Device		Do Not Use if Package is Damaged
	Manufacturer		Do Not Reuse Single-Use Device
	Date of manufacture		Prescription Only or "For Use by or on the order of a licensed medical professional"
	Lot Code		Consult Instructions For Use (IFU)
	Reorder Number		Caution
	European Conformity		Warning
	Authorized representative in the European Community		

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- Corp străin mai mare decât lățimea deschiderii cleștelui pentru îndepărtarea corpurilor străine
- Corp străin în contact cu membrana timpanică