SafeStraw™

INSTRUCTIONS FOR USE

Single patient multiple use

Rx ONLY

WARNING:
CHOKING HAZARD.
Contains small parts.


MADE IN
THE USA

This manual applies to the **REF** numbers indicated below:

#1500

#1510

#1520

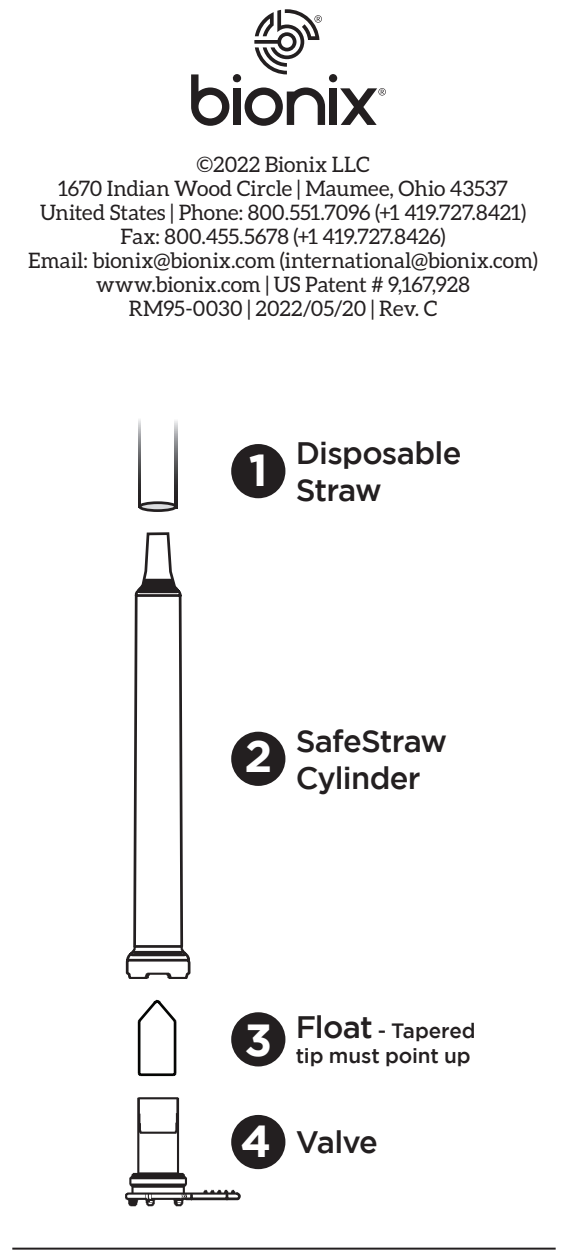Drawing 1.0
ASSEMBLY DIAGRAM

ENGLISH

SAFESTRAW™ - USER INSTRUCTIONS

Intended use: The intended use of the Safe Straw is to limit the volume of liquid the patient can draw into his/her mouth in one suck.

Indications for use: The Safe Straw is indicated for use on specific patient populations who suffer from oral motor paresis (e.g., stroke) and/or impulsivity (e.g., head injury) and are over 6 years of age.

- Choose the appropriate SafeStraw cylinder – white cylinder for thin liquids or blue cylinder for nectar thickened.
- Use one of the provided ¼" diameter flexi-straws, or one of your own. If straw is too long, simply cut to desired length.
- Press straw firmly onto the tapered end of the SafeStraw cylinder. The wide end of the cylinder is the bottom.
- Place in drink and fill the cylinder by sucking once. Suck again to begin to use. NOTE: Between sips, the float inside of the cylinder needs to reset, or fall to the bottom of the cylinder, to allow for full volume consumption on the next sip. Sink time of the float will vary depending on the thickness of the liquid (approximately 1-3 seconds for thin and 6-9 seconds for nectar thickened).
- Disassemble after use and wash in hot water and soap, or the top rack of a non-industrial dishwasher.

The Bionix SafeStraw is designed for use with thin, water like liquids, or nectar thickened liquids. Use with carbonated beverages, or honey thickened will render the SafeStraw ineffective for limiting the volume of consumption.

	Medical Device		Do Not Use if Package is Damaged
	Manufacturer		Do Not Reuse Single-Use Device
	Date of manufacture		Single patient-multiple use
	Use-By Date		Consult Instructions For Use (IFU)
	Lot Code		Caution
	Reorder Number		Warning
	Serial number		Authorized representative in the European Community
	Sterilized using irradiation		European Conformity
	Do Not Resterilize		Prescription Only or "For Use by or on the order of a licensed medical professional"
	Non-sterile		Temperature limitation

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