OtoClear® SprayWash Kit

Instructions for Use

Instructions d'utilisation du pulvérisateur Bionix	Gebrauchsanweisung Bionix Wasch-Sprühflasche	Istruzioni di montaggio del flacone per lavaggio spray Bionix
Instruções de utilização da garrafa de lavagem por pulverização Bionix	Instrucciones de uso de la botella pulverizadora de lavado Bionix	Οδηγίες χρήσης φιάλης πλύσης με σπρέι SprayWash της Bionix
Gebruiksaanstructies voor de Bionix SprayWash-fles	Bruksanvisning för Bionix SprayWash-enheten	Bionix SprayWash -pullon käyttöohjeet
Instrukcje użytkowania butelki SprayWash firmy	Pokyny k použití láhve Bionix SprayWash Bottle	„Bionix SprayWash“ buteliuko naudojimo instrukcijos
Brugsanvisning til Bionix sprøjte-skylle-flaske		

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- ENGLISH**
Contents include:
- OtoClear SprayWash bottle
 - Bottle head with wand attached
 - 20 OtoClear Tips (for #7290) or 5 OtoClear Tips (for #7295)
 - Ear basin

If any item is missing, please contact your distributor.

Intended use: The intended use of the OtoClear SprayWash Kit is to flush cerumen and/or foreign bodies from a patient's ear canal without causing injury.

Indications for use: The OtoClear SprayWash Kit is indicated for use on patients 6 months or older with impacted cerumen and/or foreign bodies occluding the ear canal where no infection and/or eardrum perforation is known or suspected.

CAUTION: DO NOT perform irrigation if the tympanic membrane is perforated or if tympanostomy tubes are present. Discontinue irrigation immediately if bleeding, irritation or other trauma to the ear canal or tympanic membrane occurs.

- WARNINGS:**
- Use only body temperature water to reduce the risk of dizziness or vertigo.
 - If pain or discharge occurs, see your physician.
 - Cross-contamination risk. Do not reuse the disposable OtoClear Tip as this may spread contamination from one patient to another patient.

Medical Device Reporting: Notice to Users and/or Patients in EU: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Clinical Benefits: Indicated in Intended Use.

- Contraindications:**
- The person has previously had complications following ear irrigation procedures
 - There is a history of a middle ear infection in the last six weeks
 - There is discharge from the ear canal or ear pain or tenderness
 - The person has or suspects an ear drum perforation
 - The person has had ANY form of ear surgery
 - The person has clef palate, repaired or not
 - The person has tympanostomy tubes
 - There is a history of inner ear problems (especially vertigo)
 - Button or hearing aid batteries are indications for emergent ENT consultation

Residual Risk: Risk associated with the use of this product has been reduced as far as possible, but the product cannot completely eliminate potential patient

MD	R_x ONLY	OTOCLEAR TIPS ARE SINGLE USE ONLY	OTOCLEAR TIPS ARE MADE IN THE USA
-----------	---------------------------	--	--

CE	EC REP	Obelis s.a Boulevard Général Wahnis 53 1030 Brussels, BELGIUM Tel: +(32) 2.732.59.54 Fax: +(32) 2.732.60.03 E-Mail: mail@obelis.net
CH REP		Obelis Swiss GmbH Ruessenstrasse 12 6340 Baar / ZG Switzerland Tel: +41 41 544 15 26 E-Mail: info@obelis.ch

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MD	R_x ONLY	OTOCLEAR TIPS ARE SINGLE USE ONLY	OTOCLEAR TIPS ARE MADE IN THE USA
-----------	---------------------------	--	--

CE	EC REP	Obelis s.a Boulevard Général Wahnis 53 1030 Brussels, BELGIUM Tel: +(32) 2.732.59.54 Fax: +(32) 2.732.60.03 E-Mail: mail@obelis.net
CH REP		Obelis Swiss GmbH Ruessenstrasse 12 6340 Baar / ZG Switzerland Tel: +41 41 544 15 26 E-Mail: info@obelis.ch

- or user harm arising from the following:
- Harm from mechanical hazards
 - OtoClear SprayWash bottle
 - Harm from misuse, or user error
 - Harm from unanticipated origins

Before you begin:
 Exam the ear canal and tympanic membrane with an otoscope noting the type and location of any cerumen. Hard or impacted wax may require a cerumenolytic agent.

Bionix SprayWash Bottle Assembly Instructions:

- Remove SprayWash Bottle (herein referred to as "water reservoir") and the spray head from packaging.
- Insert the water reservoir with body temperature water.
- Secure head with one hand, rotate the blue fastening ring at the base of the head clockwise until tight.
- Grasp the OtoClear Tip and twist onto the luer lock fitting of the adapter until it stops

- Bionix SprayWash Bottle Use Instructions:**
- Ensure the water is approximately body temperature.
 - Insert the OtoClear Tip fully into the ear canal. Note: Fully inserting the OtoClear Tip will help eliminate backplash.
 - Place a basin beneath the outer ear to capture draining effluent.
 - Pump the bottle head trigger to irrigate the ear canal.
 - Re-examine the ear canal. Repeat the irrigation procedure as needed.
 - Remove any remaining water in the ear canal with an absorbent sponge. When procedure is complete, remove and dispose of the single-use OtoClear Tip.

Cleaning and Disinfection Instructions for the OtoClear SprayWash Bottle:
 Please visit bionix.com or scan the QR Code below to view our cleaning and disinfection instructions for this device.

MD	R_x ONLY	OTOCLEAR TIPS ARE SINGLE USE ONLY	OTOCLEAR TIPS ARE MADE IN THE USA
-----------	---------------------------	--	--

CE	EC REP	Obelis s.a Boulevard Général Wahnis 53 1030 Brussels, BELGIUM Tel: +(32) 2.732.59.54 Fax: +(32) 2.732.60.03 E-Mail: mail@obelis.net
CH REP		Obelis Swiss GmbH Ruessenstrasse 12 6340 Baar / ZG Switzerland Tel: +41 41 544 15 26 E-Mail: info@obelis.ch

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©2025 Bionix LLC | 1670 Indian Wood Circle | Maumee, Ohio 43537