


OtoClear[®] Adapter Wand Instructions

For Use with OtoClear Portable Electronic Irrigator

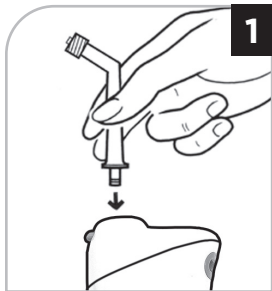

Push the top of the irrigator handle down and insert the adapter wand. Once the adapter wand is firmly in place, push the top of the irrigator handle up to 'lock' the adapter wand in position. (Make sure the Pressure Control Switch is on the lowest setting for first-time use)



Twist the OtoClear tip on the luer lock fitting of the adapter wand. (Be sure to tighten the OtoClear tip securely).

NOTE: The wand can be cleaned as needed depending on usage rate. Use a soft cloth and mild, non-abrasive cleaner.



Contents: 3 Adapter Wands.



OTOCLEAR TIPS ARE
SINGLE USE ONLY

R_x ONLY



MADE IN
THE U.S.A.



©2022 Bionix LLC

1670 Indian Wood Circle

Maumee, Ohio 43537 | United States

Phone: 800.551.7096 (+1 419.727.8421)

Fax: 800.455.5678 (+1 419.727.8426)

Email: bionix@bionix.com

(international@bionix.com)

www.bionix.com

RM95-0051 | Rev. D