

ShotBlocker®

ShotBlocker is intended to distract, comfort, and reduce the anxiety of a patient receiving minor injections and/or needle pokes.

Reorder #8002

Contents: 5 ShotBlocker® devices

MADE IN THE U.S.A.



bionix®

**Health
at Home**

PROFESSIONAL PRODUCTS FOR HOME USE

Obelis s.a • Boulevard Général Wahis 53
1030 Brussels, BELGIUM • Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03 • E-Mail: mail@obelis.net
Bionix LLC • 1670 Indian Wood Circle • Maumee, OH
43537 • United States • Phone: +1 419.727.8421
Fax: +1 800.455.5678 • www.Bionix.com



©2022 Bionix Health at Home | call 800.551.7096
fax 800.455.5678 | email bionix@bionix.com
visit 1670 Indian Wood Circle Maumee, OH 43537
United States | shop www.bionix.com
Patent# 6,902,554 | RM95-8011 | Rev. C

Instructions for Use

1. Remove one ShotBlocker® from its package.
2. Select the injection site and prep the skin as instructed by your medical professional.
3. Press ShotBlocker firmly against the skin so that the blunt contact points touch the skin at the injection site.
4. Immediately administer the injection in the usual manner through the central opening of ShotBlocker.

NOTE:

- a. Do not move or remove ShotBlocker until the injection has been completed.
 - b. If more than 20 seconds elapse between the placement of ShotBlocker and the injection, completely remove ShotBlocker from the skin and repeat the process beginning with Step 3.
5. Dispose of the needle and syringe in the proper container.
 6. Clean ShotBlocker with mild detergent and water and allow to dry.



Cleaning Instructions

After each use, clean ShotBlocker with mild detergent and water. Dry completely.

Indications for Use

ShotBlocker is designed to distract and comfort patients receiving intra-muscular or subcutaneous injections and needle pokes.

WARNINGS/Cautions

Intended to be used for single user. **DO NOT** share between users.